


Daily Patients Page Overview

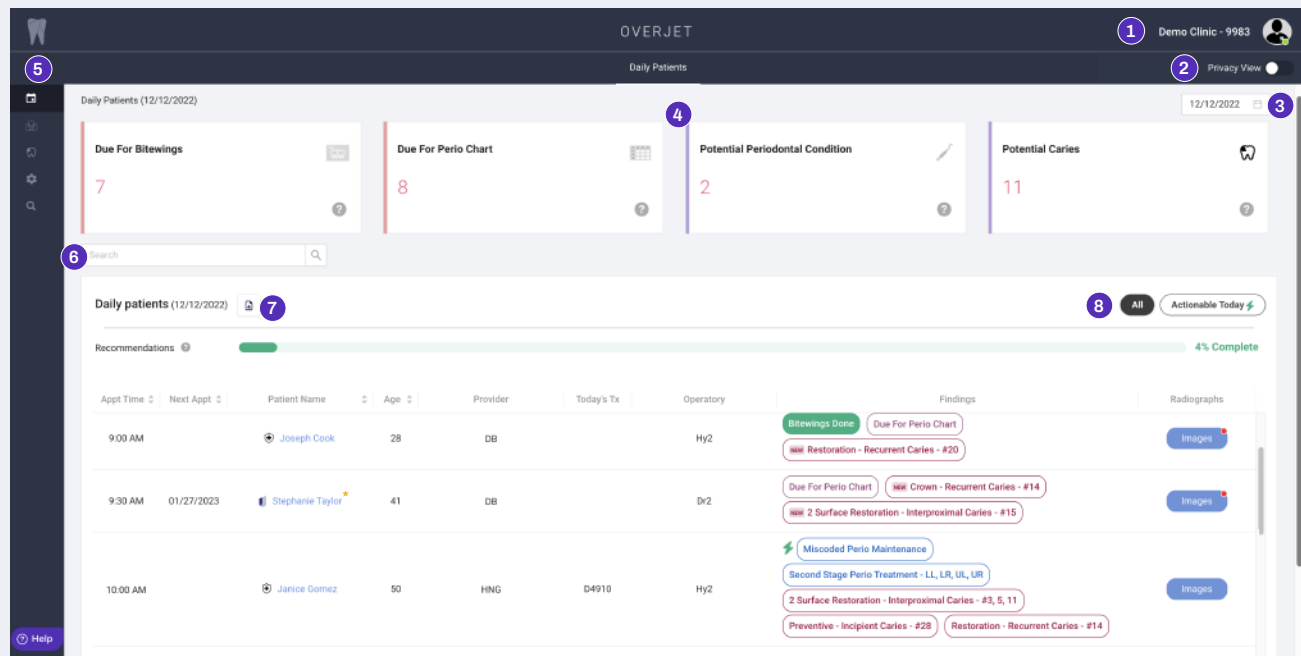


Log in via the Overjet taskbar icon or clinic.overjet.ai

- 1 Clinic ID: If you have access to multiple locations, click the silhouette and the options will show in a dropdown menu.
- 2 Privacy View: Toggle to change the patient names to patient ID numbers to protect PHI.
- 3 Calendar: This defaults to today's date. You may select other dates to view by clicking on the desired date.
- 4 Filter Cards: These cards at the top can be used to filter the Daily Patients page for specific criteria.
- 5 Left Navigation Bar: Hover over the icons to see the names for each one.

Daily Patients	<ul style="list-style-type: none"> • Daily Patients: A list of patients on today's schedule.
Periodontal	<ul style="list-style-type: none"> • Periodontal: An overview of patients who may be candidates for periodontal treatment.
Restorative (Beta)	<ul style="list-style-type: none"> • Restorative: An overview of patients who may be candidates for restorative treatment.
Configuration	<ul style="list-style-type: none"> • Configuration: Admin access for configuring operatories, providers, session settings, and visualization settings.
Patient Search	<ul style="list-style-type: none"> • Patient Search: A search bar to search the database for the past 18 months by patient name, ID, or date of birth.

- 6 Search Bar: Type in the operatory name or provider name to filter the list by the desired criteria. You may also search for a patient name or patient ID on today's schedule.
- 7 Print button: Click this icon to print the Daily Patients page.
- 8 All: A list of all Overjet findings

Daily Patients Page Overview, Cont.

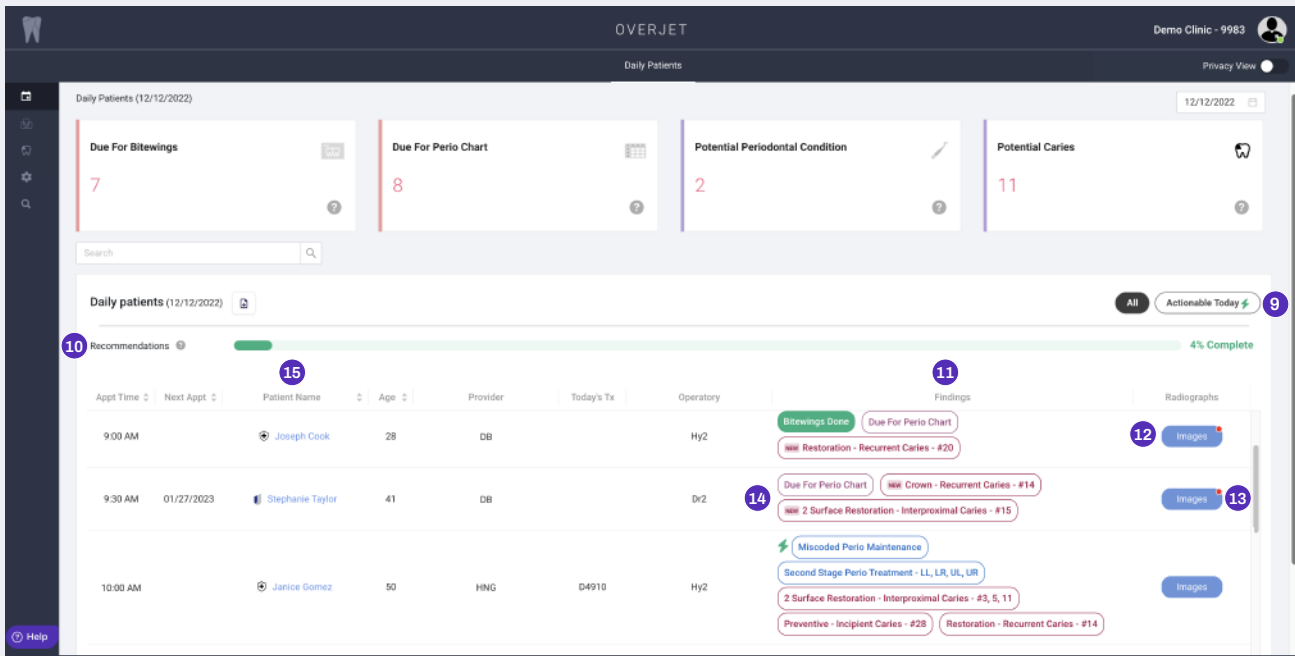

- 9 Actionable Today: Click to view a list of potential findings for your consideration that Overjet has identified as being able to address at today's appointment.
- 10 Recommendations: This green bar shows the percentage of recommended Overjet findings that have been completed.
- 11 Findings column: A list of potential treatment needs that have not been diagnosed yet and of patients who may be due for periodontal probing and/or radiographs.
 - Findings that are completed will be automatically updated as Overjet syncs with the digital imaging system and practice management system. They will show as green.
 - To track why a finding was not completed, click on the finding to bring up a list of reasons and select the most appropriate. The list of reasons changes based on the type of procedure.
- 12 Images: Click this button to access the patient's radiographs taken within the past 18 months. New images taken today will appear within a few minutes after saving them in your imaging software.
- 13 Red dot: Signifies that radiographs taken today are ready, but have not been looked at yet from this workstation.
- 14 Findings Legend:
 - Green: Indicates the action has been completed.
 - Purple: Indicates the patient may be due for radiographs or periodontal charting.
 - Red: Indicates AI findings of potential restorative needs.
 - Blue: Indicates AI findings of potential periodontal needs.
 - Yellow: Indicates some recommendations have been addressed, but not all.
 - Gray: Indicates a comment has been entered explaining why not completed.
 - NEW New: Indicates the finding is from an image taken today.
 - ⚡ Lightning Bolt: Indicates an action that Overjet identified as being able to address at today's appointment.
- 15 Patient Name: Click the patient's name or Patient ID if in privacy view to pull up that patient's transaction history.

Daily Patients Page Overview, Cont.

The screenshot displays the OVERJET interface for the 'Daily Patients' page. At the top, there are four summary cards: 'Due For Bitewings' (7), 'Due For Perio Chart' (8), 'Potential Periodontal Condition' (2), and 'Potential Caries' (11). Below these is a search bar and a 'Daily patients (12/12/2022)' section with a '4% Complete' progress bar. The main table lists patients with columns for Appt Time, Next Appt, Patient Name, Age, Provider, Today's Tx, Operatory, Findings, and Radiographs. Three patients are highlighted with callouts: Joseph Cook (age 28, DB provider), Stephanie Taylor (age 41, DB provider, marked with a gold star), and Janice Gomez (age 50, HNG provider). A 'Help' icon is visible in the bottom left corner.

Appt Time	Next Appt	Patient Name	Age	Provider	Today's Tx	Operatory	Findings	Radiographs
9:00 AM		Joseph Cook	28	DB		Hy2	Bitewings Done, Due For Perio Chart, Restoration - Recurrent Caries - #20	Images
9:30 AM	01/27/2023	Stephanie Taylor	41	DB		Dx2	Due For Perio Chart, Crown - Recurrent Caries - #14, 2 Surface Restoration - Interproximal Caries - #15	Images
10:00 AM		Janice Gomez	50	HNG	D4910	Hy2	Miscoded Perio Maintenance, Second Stage Perio Treatment - LL, LR, UL, UR, 2 Surface Restoration - Interproximal Caries - #3, 5, 11, Preventive - Incipient Caries - #28, Restoration - Recurrent Caries - #14	Images

- 16 ★ Gold Star: This indicates the patient is a new patient.
- 17 Icons: Symbols to the left of a patient's name indicate the type of insurance.
- 18 Age: You will be able to view images for patients under 13, but they will not include findings.
- 19 Help: Open the help desk widget to submit issues and questions, along with screenshots if needed.

Get in Touch. practicesupport@overjet.ai clinic.overjet.ai