

Periodontal Page Overview

The screenshot shows the Overjet Periodontal Page interface. At the top, the header includes the Overjet logo, the text "OVERJET", the practice name "Balaban Dental - 1234", and a user profile icon. Below the header, there's a "Daily Patients" section with a "Privacy View" toggle (callout 4). The main content area is divided into several sections:

- PATIENTS POTENTIALLY IN NEED OF TREATMENT**: This section contains four cards:
 - Scaling And Root Planing** (callout 1): 67 patients, \$12,014.
 - Possible SRP** (callout 2): 10 patients, \$1,090.
 - Miscoded Perio Maintenance**: 0 patients, \$0.
 - Second Stage Periodontal Treatment**: 177 patients, \$5,902.
- POTENTIAL PERIO EVALUATIONS**: This section contains four cards:
 - Periochart Missing**: 42% (657 out of 1547 active ADULT patients).
 - Miscoded Perio Evaluations**: 1 patient (Coded for D0150).
 - Radiograph Missing**: 11% (201 out of 1765 active patients).
 - Radiographs Not Billed**: 67 patients, \$5,902.
- THIRD MOLAR REVIEW (BETA)**: This section contains one card:
 - Third Molar Review**: 94 patients.

Below these sections is a table with 8 columns: Scaling and Root Planing, Possible SRP, Miscoded Perio Maintenance, Second Stage Perio Treatment, Periochart Missing - Adult Patients, Radiograph Missing, Radiographs Not Billed, and Third Molar Review. The table lists 67 patients (callout 5). The first patient, Alexandra Moulaison, is highlighted. The table includes columns for Name, Provider, Quadrants, Pocket Depth, Bone Level, Sub-Gingival Calculus, and Radiographs (callout 6). A Help icon (callout 7) is located at the bottom left of the table.

1 Access the Periodontal Page by clicking on the icon in the left sidebar

2 Clickable Filter cards to quickly see patients with specific criteria in the table below

3 Clickable Question Mark icons provide guidance for periodontal card criteria

4 The privacy-view toggle allows you to change the patient name column to display patient ID to protect PHI. Once you click on your images, the patient name will display, even with the PHI filter on

5 Clickable names that will take you to patient timeline

6 The Images Button is where you will find radiographs taken within the past two years. New images taken today will appear within approximately three minutes after saving them to your imaging software. Click the images button to view the radiographs with Overjet's AI findings

7 Help icon that opens up the help widget. You can submit issues and questions, along with screenshots of the page you are viewing to our customer support team.

