

Restorative Page Overview

The screenshot shows the OVERJET interface for Balaban Dental - 1234. The top navigation bar includes 'Daily Patients' and a 'Privacy View' toggle. The main content area is divided into two sections: 'POTENTIAL TREATMENTS' and 'LIST OF PATIENTS POTENTIALLY NEEDING TREATMENT'. The 'POTENTIAL TREATMENTS' section features seven filter cards for different dental procedures, each showing a patient count and a total cost. The 'LIST OF PATIENTS' section is a table with columns for treatment criteria, patient name, provider, quadrants, tooth number, and radiographs. A 'Help' icon is located in the bottom left corner of the sidebar.

1 Access the Restorative Page by clicking on the icon in the left sidebar

2 Clickable Filter cards to quickly see patients with specific criteria

3 Clickable Question Mark Icons provide guidance for Restorative Criteria

4 The privacy-view toggle allows you to change the patient name column to display patient ID to protect PHI. Once you click on your images, the patient name will display.

5 Click the “Images” button to view the radiographs presenting with selected criteria

6 List of patients who meet criteria selected by clicking the cards above

7 Help icon that opens up the help widget. You can submit issues and questions, along with screenshots of the page you’re viewing, to our customer support team.